BEAUTIFLY

Meso
P R O
series

Dermapen
Dermapen

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 /beautifly.eu  @beautifly.eu  beautifly.eu

 /beautiflyeu  @beautifly.eu

f1

f2

f3

 L1

 R3-M

 R5

 F3

 F5

 F7

 F9

 9

 12

 24

 36

 42

 Nano

 Nano

 3DNano

 SNano

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- www.beautifly.eu
- www.youtube.com/beautifly.eu

Adapter:	5V-1000mA
Package weight:	318g
Package Size:	169*123*56mm
Included:	stand, adapter, manual, power cable, 2x needle cartridge

Security:

Before using the product, read this user manual and follow the recommendations contained therein when using the device.

Part description [page 2 f1]

1. Needle cartridges
2. Adjustment ring
3. Power/speed button
4. Power

How the device works:

Dermapen Meso Pro can be used both with a built-in battery and with mains power.

1. Plug the DC adapter into the socket
2. Plug the DC adapter into a power outlet
3. To charge the battery: plug the adapter into the socket and connect it to the power interface

Attention:

The power indicator remains red while charging
When charging is complete, the power indicator will turn green

How to use:

1. Remove the needle from the sterile packaging.
2. Aligning the round slot, push the needle cartridge horizontally until it stops.
3. Lock the needle cartridge by turning it counterclockwise.
4. Rotate the needle cartridge clockwise, then push it out horizontally to remove it.
5. Turn on the device by pressing the power button for 2-3 seconds to turn on or turn off the device.

Needle length adjustment:

Adjust the depth of the needles by turning the scale adjustment ring. Turn clockwise, the length of the needles will be shorter. Turn to the left, the length of the needles will be longer.

5 speed levels adjustment:

Dermapen Meso Pro, works at 5 different speeds
The speed changes one higher each time you press the power button.

Matching needle cartridges [page 3, f3]**Indications**

Microneedle mesotherapy treatment is used to revitalize and rejuvenate the skin of the face, neck and cleavage. It can also be successfully used on the skin of the hands to improve the density of thin and flaccid skin. Dermapen Meso Pro is also suitable for rejuvenating the eye area. It reduces wrinkles and reduces the size of pores. The method is also used to reduce cellulite and stretch marks. Dermapen Meso Pro brings significant results in the treatment of baldness and hair loss, as well as the correction of acne and post-operative scars and post-traumatic.

Contraindications

- active inflammation and purulent processes during the procedure
- pregnancy and breastfeeding
- unregulated diabetes
- unregulated thyroid diseases
- tendency to develop keloid scars
- allergy to any of the ingredients of the substances used for the treatment
- in people with vascular skin
- use of drugs that reduce blood clotting
- active cancer diseases
- use of anticoagulant drugs (e.g. aspirin)
- fever

Precautions

- Do not use it for power other than AC 220V. Otherwise, it may cause fire or malfunction
- Do not use it near high temperature, dusty or humid place (near the humidifier), it may cause malfunction.
- Do not repair, disassemble or modify the controller and body. This may cause a malfunction.
- Do not touch the device or unplug the power cords with wet hands. This may cause electric shock.
- Do not use stripped wire or supported wire. This may cause a fire (remember to replace it with another one).
- Do not use if the cord is cracked or damaged. May cause skin irritation.
- If you do not use it for a long time, please unplug it from the socket.
- If in doubt, please contact the seller.

Storage

- Store the device with the power turned off.
- Wipe its surface with a dry cloth after each use.
- Protect from direct sunlight or high temperature and humidity.
- Do not attempt to repair the device yourself.
- Please check the device and adapter before use.
- When using a device that has not been used for a long time, check whether it is working properly and safely.
- Keep out of reach of children and pets

More information about the company and products [page 2 - f2]:

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[EN] Rules for handling electrical and electronic equipment: Electrical and electronic equipment as well as batteries and accumulators must not be combined with other household waste. This is indicated by the crossed-out wheeled bin symbol on the packaging. Dispose of waste electrical and electronic equipment as well as batteries and accumulators at your local separate collection point or to your dealer. Electrical and electronic equipment and batteries and accumulators may contain hazardous substances, mixtures and components that are harmful to the environment and human health. Incorrect handling or damage to them may lead to damage to health or environmental pollution when the equipment is further disposed of or recycled. The device was placed on the market after August 13, 2005

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[EN] Summary declaration of conformity: The manufacturer hereby declares under its sole responsibility that the above product complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on the harmonisation of the laws of the Member States relating to electromagnetic compatibility and Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment, together with Commission Delegated Directive (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances.

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Importer: Beautifly Sp. z o.o. Plac Bankowy 2,

00-095 Warszawa, Polska

Model: Meso Pro